

STEAMGOOGLE : POWERFUL NEW MOIST HEAT TREATMENT FOR DRY EYE RELIEF



Utilizes Innovative Heater Technology



Safe and Effective Dry Eye Relief w/ No Pressure Applied to the Cornea



Easy Setup and Use



Fast Warm Up Time



Pinpointed Moist Heat Therapy


Unclogs Oil Glands for Relief From Symptoms of MGD/Blepharitis/Styes/Chalazions/Abrasions

Read/Watch TV During Treatment

Use Anywhere With The Included Portable Power Bank

A MORE EFFECTIVE DRY EYE TREATMENT

The patented SteamGoggle is an easy to use at home dry eye treatment device that uses moist heat to alleviate symptoms caused by MGD and Blepharitis. This concentrated moist heat combined with the eyes own blinking motion, serve to more effectively unclog the eyes oil glands and help restore their function. The device is easy to setup (simply add water to each eyecup and press power button) and clean (place entire goggle under faucet to rinse), while also allowing the user to read/watch TV during treatment. Innovative technology including PTC heaters with embedded safety features and a high performance/heat-resistant PESU plastic lens, allows the device to heat up fast and precisely, which allows for a high concentration of steam for each treatment. This sustained high concentration of moist heat (50°-55° celcius-may take some users several treatment sessions for their eyes to acclimate to) allows the device to provide a more effective dry eye treatment compared to other methods. Use the device anywhere, anytime when using the included portable power bank, with over 15 treatments per charge.

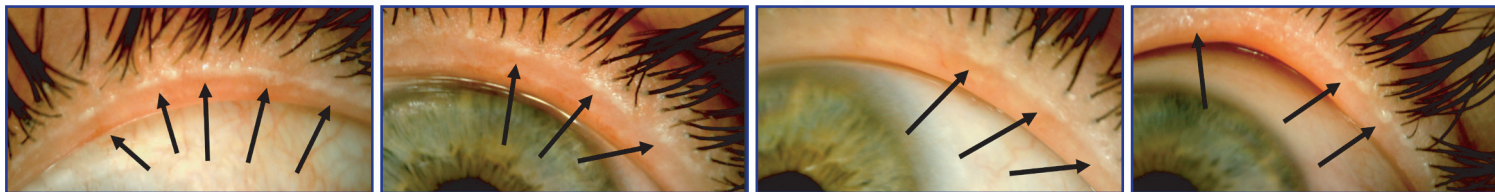

The device works by combining moist heat w/ blinking to warm/soften and then massage/remove solidified oil gland contents. Thickened, white oil gland contents can be visible on lid margin following treatment (see images of white residue left on inventor's lid margin above).

FEATURES

- ① **COMBINES MOIST HEAT WITH BLINKING TO MORE EFFECTIVELY UNCLOG THE EYES' OIL GLANDS AND PROVIDE RELIEF OF SYMPTOMS CAUSED BY MGD/BLEPHARITIS/STYES/CHALAZIONS:** The patented SteamGoggle is an easy to use at home dry eye treatment device that combines moist heat of adjustable temperature with blinking to alleviate dry eye symptoms.
- ② **POWERFUL/CONSISTENT MOIST HEAT WHILE LOOKING AT PHONE/TV:** Sustained high concentration of moist heat (50°-55° celcius), combined with the eyes own blinking motion, allow the device to better unclog the eyes' oil glands and help restore their function.
- ③ **USE ANYWHERE AND FOR AS LONG AS YOU WANT WITH INCLUDED POWER BANK:** The included USB-C power bank (with over 15 treatments per charge) and mesh carrying bag allow you to easily take and use the device anywhere. Easily extend the 8.5min treatment time to your preference (i.e. 20-30mins) by refilling water lattice and restarting timer.
- ④ **EASY TO USE/CLEAN:** The device is easy to setup (simply connect to powerbank, add water to each eyecup and press power button to begin) and clean (place entire goggle under faucet to rinse).



*Power Cable Not Shown

ABOUT

Starting in late high school and continuing through college and my working life, I have suffered through a continuous bout of symptoms related to dry eye, involving a nonstop gritty, scratchy, foreign body sensation in my left eye.

After years of finding no relief from current products/treatments on the market, (being a mechanical engineer) I created a new device that finally ended up alleviating my symptoms. The SteamGoggle differs from other products on the market, as it uses a pinpointed approach to deliver effective amounts of sustained moist heat directly to the eyes, all in a safe, easy to use at-home package.



Intro/Guide Video

See website or scan barcode with smartphone camera to view the short video




info@steamgoggle.com

www.steamgoggle.com

Instructions for Use

- 1** With clean hands, begin by adding water (we recommend purified) to the plastic lattice grooves (or squares) inside of each eye cup by using the squirt bottle or by pouring water into the eyecup. Note that treatment temperature can be affected by filling more or less grooves with water. Filling them all will give the maximum temperature, while filling none will give the lowest.

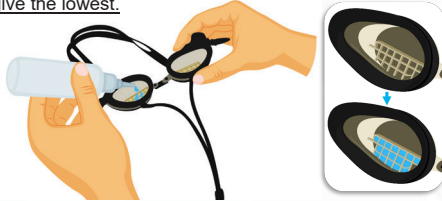

- 2** Tilt the goggles upside down and lightly tap with finger to remove any excess water. Some water should remain in the plastic grooves.



- 3** Connect the device to electric power, via a portable power bank using its USB-C 12 volt output port. There should be an audible beep from controller. Next, activate the heater by pressing the power button.

Power Bank



- 4** Allow the device to pre-heat for 1 minute 30 seconds to allow device to come to temperature. Power switch will emit an audible double-beep to notify user that pre-heating has finished.

Preheat



- 5** Again, tilt the goggles upside down and lightly tap with finger to remove any excess warmed water. Some water should remain in the plastic grooves. Note that as the central area around the heater may be warm to the touch, it is advised to handle the goggles by the outer top and bottom edges.




- 6** Put the goggles loosely over the eyes to begin treatment. The lens may initially be cloudy, but the bottom section should shortly defog to the point where the user can look at their phone/TV.



- 7** At the end of the 8 minute 30 second treatment time, the power switch will triple-beep and shut off power. If desired, treatment can be extended by restarting power button and refilling plastic grooves with water if it has evaporated.



- 8** Clean the eyelids and device with clean water and a mild cleanser (i.e. we recommend baby shampoo) after each treatment session. Note that the entire goggle is water resistant (front and back), and can be placed under the faucet to easily wash away any soap remnants. Just ensure that water does not get on the power switch or power bank.

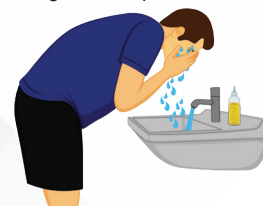

IMPORTANT: Treatment must be performed while sitting up with face looking down or straight ahead. Do not look up or lay head down during treatment to ensure heated water is retained in eyecup.